**Security Fixing Kit Installation Guide**

1. Fit the angled bar (B) onto the foot, and then screw the nut (C) tightly against it.
2. Refit the feet to the step ensuring the nut (A) is tight against the Adjusting Leg.

Please read these instructions along with the instructions on page one. Ensure the step is stable before use. Periodically check and retighten all nuts or other fixings as required.

The angled bars (B) should be fitted to diagonally opposite corners. At least 15mm of thread must remain inside the adjusting leg for safe use.

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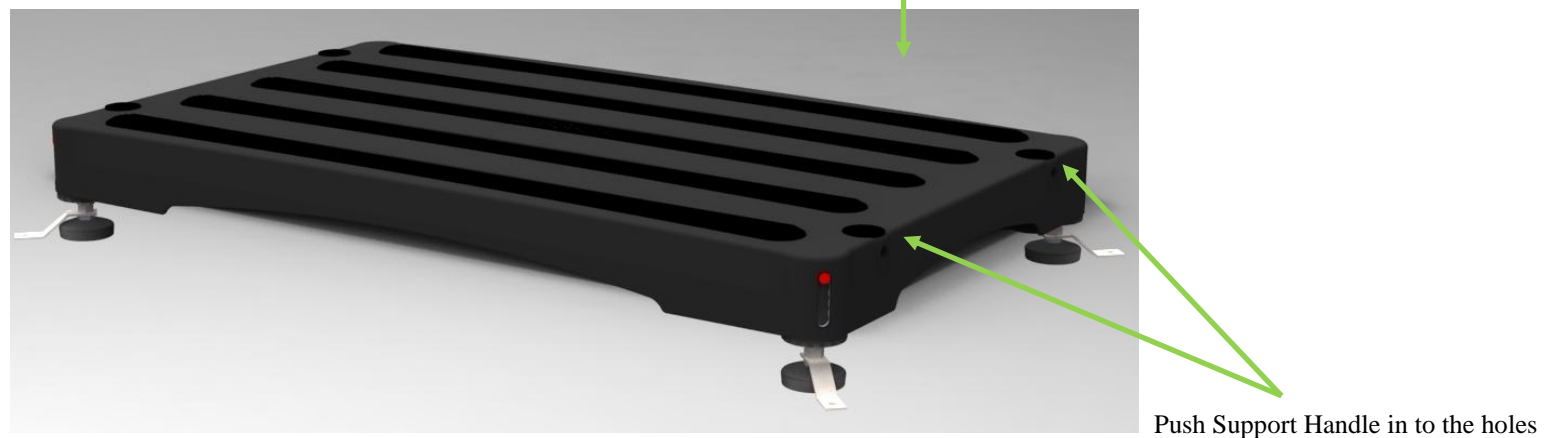
### Fitting the Support Rail(s)

## ENGLISH

The Support Handle is designed to assist people who are unsteady on their feet to achieve a more secure transfer using the step. If required a Support handle can be fitted to both sides of the step. It is to be used only when fitted to a Bigfoot Half Step.

Install the Security Fixing kit; do not use the Support Handle without installing the Security Fixing kit. Please refer to the installation guide shown below.

- To fit remove the two covers at the side of the step, press the two buttons at the base of the handle and insert in to the openings.
- Before use check the step is secure.
- Tread centrally onto the step surface avoiding stepping on to the corners or along the edges of the step, do not lean backwards as this is less stable for the user.

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